

General Calibrator

Volume: 3 ml

Material: Lyophilized

Preface:

General Calibrator is a lyophilized calibrator which is prepared from human serum. The analytes values have been selected with the aim of accurate and reliable control of automatic analyzers.

Preparation:

Open the vials lid Carefully and be aware not to lose the contents of the vial.

Add 3 ml distilled water to the vial and close the lid tightly.

After 10 minutes at the temperature of 2-8 °C, upside down the vial and wait for 10 minutes again. During this time, protect the vial from direct light and invert it several times gently then return it to the first position.

Avoid shaking the vial violently and creating foam.

Storage and Stability:

All the components of vial (lyophilized) are stable until the expiration date on the label at 2-8°C.

After preparation, analytes are:

stable for eight hours at 25°C,

stable for five days at 2-8°C,

stable for twenty days (only freezing once) at -20°C, except for the following:

Test Name	Stability after preparation
Total Bilirubin	10 days
Direct Bilirubin	10 days
AST	10 days
ALT	10 days

Warning and precautions:

This product is only for laboratory use.

Add: NO.68, Sorena St., Damavand 2 Industrial Park, Damavand, Tehran, IRAN

Tell: 021-77778007-8

website: www.pts-ico.ir

Fax: 021-26142195

email: pts.ico@gmail.com

V.F1